



Momitalk Quick Operation Guide

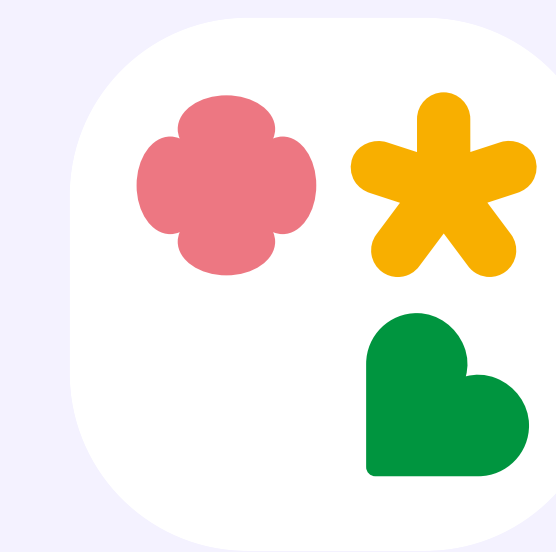

To start and finish recording

- 1 Please check the status of the MomiBox.

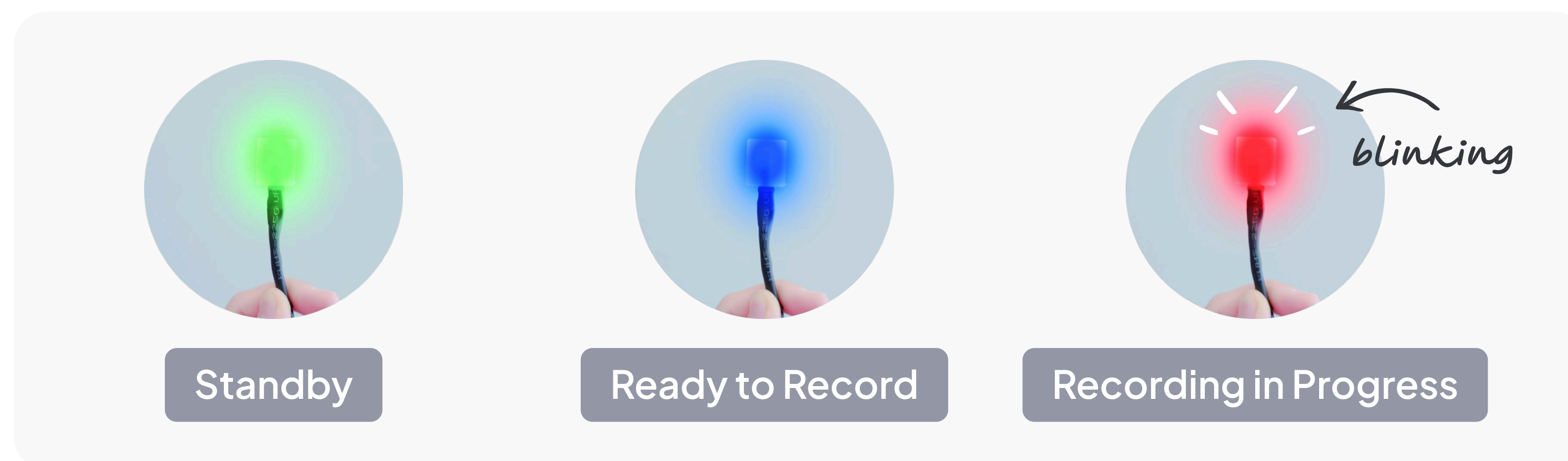

- 2 Scan barcode.
LED turns ● **Blue**, speaker prompt: "Ready to Record"
- 3 Start motion on the ultrasound to begin recording.
LED turns ● **Red**, speaker prompt: "Recording"
- 4 Scan Finish QR.
LED turns ● **Green**, prompt: "Recording Ended"
→ The upload will automatically start.

Other Functions during recording

Pause/Resume the recording by scanning QRs as needed.

"Pause" - LED ● **Blinking Blue**

"Resume" - LED ● **Blinking Red**

Capture photos by pressing the button on the photo clicker remote/ wired button.

A "beep" sound will be heard from the speaker confirming each capture.

To verify upload & Hide/Unhide content

Check Admin Portal (upload speed depends on internet speed).

Admin Portal
admin.momitalk.com

Order supplies (barcodes/brochures)
support.momitalk.com/materials

Knowledgebase (support articles)
support.momitalk.com

Momitalk Support (M-F 9am-5pm PT)
us_help@momitalk.com

Momitalk Quick Start Guide (Full)
support.momitalk.com/quickstart

Update WiFi on Momtalk Device
support.momitalk.com/wifi